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GB I NOTE!

Dear customer, please read the safety instructions carefully before use and keep the manual for further use. The manufacturer assumes no liability for faulty assembly or improper use of the product.
Safety instructions: The fold-down shower seat is used exclusively as a seat during showering and in private use. The mounting height of the seat depends on the body height of the user, as standard height is used 450-550mm (17.1"-21.65") from the floor of the showertray. As a simple method the mounting height of the shower seat we recommend the following: Measure the seat height of a chair comfortable to the user and mount the shower seat at a similar height. Do not stand on the seat. Keep packaging materials away from children. Moist surfaces can impair the anti-slip functi-on. The recommended maximum usage is 5 years. Do not use if the product is damaged.
Cleaning and care instructions: Please pay attention to the cleaning and care instructions. In the case of improper use the product can be damaged. Clean it carefully with water and standard commercial detergents. Subsequently, you can dry the product with a cloth.

⚠ WARNING!

The installer must ensure that the construction of the wall guarantees a secure attachment of all wall mounting screws. The assembly must in any case be carried out on a stable, load-bearing wall with a smooth surface. Fasteners: dowels 8 x 65mm (0.31 x 2.56 "), screws made of stainless steel 5 x 75 mm (0.2 x 2.95 "), suitable for reinforced concrete or solid brick M20. For other types of wall, a specialist firm of fastening technology has to be consulted.

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